

ONEMED DISPOSABLE SYRINGE

ONEMED Disposable Syringe is a single-use syringe designed to support safe, sterile, and accurate medical procedures. Manufactured from high-quality materials and sterilized through standardized processes, it helps minimize the risk of contamination while providing convenience and reliability for healthcare professionals in daily practice. The product is available in a wide range of sizes, with or without needles, as well as various connector types to accommodate diverse clinical and medical applications.

- Sterile
- Single Use Only – Designed for single use to prevent the risk of cross-contamination
- Non-Toxic & Non-Pyrogenic
- Available in Luer Slip, Luer Lock, Catheter Tip, and Eccentric Tip configurations

Technical Information

- Material: Medical-grade Polypropylene & Silicone
- Sterilization Method: Ethylene Oxide (EO)
- Packaging: Individual sterile blister pack
- Manufacturer: PT. Jayamas Medica Industri Tbk

Instructions for use

- Secure the needle with the syringe barrel, turn it clockwise.
- Pull the needle cover, syringe ready for use.

Caution

To avoid cross-infection due to an accidental puncture:

1. Do not re-cap used syringes.
2. Dispose of used syringes, needles, and caps to the medical waste disposal site (sharp container / safety box).
3. If the needle still needs to be re-capped, place the cap on the table, then insert the needle (without having to hold the cap). After the needle is closed, tighten the cap until it locks.
4. Follow your institution's procedures for safe handling of sharps.
5. Check the packaging properly and read the instructions before using the device.

Plant Krian, Sidoarjo

Sebelah Selatan Jalan By Pass RT.005/RW.001, Sidomojo, Krian,
Sidoarjo, Jawa Timur, Indonesia
☎ +62.31.8982349 / +62.31.8985269

Plant Mojoagung, Jombang

Dusun Karangmenjangan RT.001/RW.001, Karangwinongan,
Mojoagung, Jombang, Jawa Timur, Indonesia
☎ +62.321.4890907

Website :

www.onemed.co.id

E-mail :

onemed@onemed.co.id



ISO 13485 : 2014
Cert No : 01 05 B 24000134

Specifications



OneMed Disposable Syringe with needle

Volume (mL)	Needle Size	Tip Type
1	26G x 1/2"	Luer Slip
1	27G x 1/2"	Luer Slip
2,5 and 3	23G x 1 1/4"	Luer Lock
3, 5	18G x 1 1/4"	Luer Lock
5	22G x 1 1/2"	Luer Lock
10	21G x 1 1/2"	Luer Lock



OneMed Disposable Syringe without needle

Volume (mL)	Tip Type
3, 5, 10, 20, 50	Luer Lock
20, 50	Luer Slip
50	Catheter Tip

Plant Krian, Sidoarjo

Sebelah Selatan Jalan By Pass RT.005/RW.001, Sidomojo, Krian,
Sidoarjo, Jawa Timur, Indonesia
+62.31.8982349 / +62.31.8985269

Plant Mojoagung, Jombang

Dusun Karangmenjangan RT.001/RW.001, Karangwinongan,
Mojoagung, Jombang, Jawa Timur, Indonesia
+62.321.4890907

Website :

www.onemed.co.id

E-mail :

onemed@onemed.co.id



ISO 13485 : 2014
Cert No : 01 05 B 24000134